

Tru-Frecator

Model TF-300

**ELECTRO FOR COAGULATION
DESCICCATION
FULGURATION**

STANDARD ACCESSORIES

TF-301 Foot switch

TF-302 Needle holder

TF-303 2 set of needles



Introducing the Tru-Frecator Model TF-300

What are skin growths? Skin growths are accumulations of various types of cells that look different from the normal surrounding skin. They may be raised or flat and range in color from dark brown or black to flesh-colored or red. They may be present at birth or develop later as we age.

Are they safe to remove?

In most cases, skin growths are non-cancerous and can be safely removed. The treatment is performed by a professional, certified technician who has been trained to identify the different types of skin growths appropriate for the treatment. All treatments are performed using the strictest sanitary procedures.

What is the Tru-Frecator?

The Tru-Frecator is an electrical treatment device that has been successfully used for over 30 years by skin care therapists to remove common surface skin growths.

How does the Tru-Frecator work?

The device uses high-frequency electrical current that delivers precise electrical sparks through a micro needle electrode at different amplitudes to dehydrate a skin growth. At almost all times, the needle remains above the skin surface and the current sparks across the gap causing dehydration of the skin growth. As the growth is removed, a natural protective coating of scab develops that covers the treated area. There is no blood associated with the treatment.

Once the scab drops off, the treated area will heal and a new healthy skin will return and gradually take on the color of the surrounding tissue. In most CMOS, there is no visible indication of the previous growth.

How does it differ from liquid nitrogen?

The Tru-Frecator Model TF-300 is generally more effective than liquid nitrogen in removing common skin growths. Frequently, more than one application of liquid nitrogen is needed to remove a growth, whereas only one Tru-Frecator treatment is ever necessary. In addition, Tru-Frecator treatments are less expensive.

Is the treatment permanent?

Yes. Once a skin growth has been treated with the Tru-Frecator Model TF-300, it does not return. However, if your skin is prone to these types of growths, new ones may appear elsewhere on your face and body.

**NOW, A SAFE, NON-INVASIVE
TREATMENT THAT PERMANENTLY
REMOVES COMMON SKIN
IRREGULARITIES FROM YOUR FACE
AND BODY IN JUST ONE SESSION**

